

Cornea Procedure Instruments



DMEK



E3121 Straiko Twin Ring Forceps

Remove diseased Descemet's membrane without disrupting the stromal bed. The ringed tip design enhances fine grasping of the membrane without catching stromal fibers that may promote graft detachment. Overall length: 82.55 mm, 3.2 inches.



E3127 Sales Stripper Paddle

This double-ended instrument features a standard Terry reverse Sinskey hook and paddle for Descemet's membrane removal. Both tips easily fit through a 1.0 mm side port incision. The curved tip of the paddle wipes the Descemet's membrane from the patient's cornea without disrupting the stroma. The tip's textured surface engages the Descemet's membrane in cases of Fuchs dystrophy or pseudophakic bullous keratopathy. Use the mirrored surface of the paddle to inspect for loose tags of Descemet Membrane. Overall length: 119 mm, 4.7 inches.



E3128 Fogla DM Stripping Hook

Designed for removing unhealthy Descemet's Membrane (DM) during DSEK, DSAEK and DMEK surgery. It features a round, ball-shaped tip bent upwards at a 90° angle. The tip is designed to prevent damage to corneal stromal fibers when pressed against the cornea to break DM. It also ensures the inner surface of the cornea remains smooth during this process. Overall length: 114.3 mm, 4.5 inches.



E4917 Straiko DMEK Cannula

Designed to assist in unrolling the donor tissue by tapping the cornea in DMEK surgery. The cannula also allows adjusting of anterior chamber depth with the addition and removal of fluid. Overall length excluding hub: 23mm, 0.9 inches.



E4918 Fogla DMEK Cannula

Designed to assist unrolling the Descemet's membrane after insertion in the patient's eye. The small gauge allows it to be slipped in tight tissue rolls and the cross ports facilitate controlled bursts of fluid laterally to open the scroll. 27 gauge. Overall length excluding hub: 23mm, 0.9 inches.



DSEK



9-781W Corneal Marker 8.0mm

Provides a template for scoring and stripping of Descemet's membrane during the DSEK procedure. Overall length: 122mm, 4.8 inches.



E2015 Snyder Stripping Forceps

Complete the stripping technique with angled tips to grasp the edge of Descemet's membrane. Iris stop allows the surgeon to reliably grasp the membrane without grasping iris or other ocular structures. Overall length: 83mm, 3.3 inches.



E2039 Carlson DSEK Graft Forceps

Complete the stripping technique with angled tips to grasp the edge of Descemet's membrane. Iris stop allows the surgeon to reliably grasp the membrane without grasping iris or other ocular structures. Overall length: 83mm, 3.3 inches.



E2040 Carlson DSEK Smoother

Press the ball-shaped tip on the corneal surface after the graft is implanted to remove fluid between the recipient bed and the DSEK graft. Overall length: 122mm, 4.8 inches.



E3050 6.00mm-9.50mm Tanne Universal Trephine Blades

A unique universal hub diameter with varying blade edge diameter provides an easier, more secure means of mounting the blade to the handle. Designed for use with: E3054 Iowa P.K. Press Punch. Packaged sterile. Single-Use. Overall length: 17mm, 0.7 inches. 1/box.



E3054 Iowa P.K. Press Punch

Precisely punch the donor corneal button. Blade holder is designed for use with the E3050 series disposable trephine blades. Unique two color cutting block aids in centration of the donor tissue. Recessed base assures that the block is held centrally under the trephine blade. Supplied with two cutting blocks (E3054 B).





G. E3113 Devers-Terry Dissector Straight

Used to dissect or separate stromal tissue during a DLEK procedure. Overall length: 127mm, 5.0 inches



H. E3114 Devers-Terry Dissector Curved

Used to dissect or separate stromal tissue during a DLEK procedure. Overall length: 127mm, 5.0 inches.



I. E3119 Reverse Sinskey Hook

Used to begin the dissection in DSEK surgery by scoring Descemet's membrane and then to gently strip the membrane. Overall length: 117mm, 4.6 inches.



J. E3122 Nick Pick #2

Used to aid in positioning the donor corneal tissue. Overall length: 115mm, 4.5 inches.




K. E3123 Terry Scraper Irrigating

Used with irrigation to stabilize the anterior chamber while gently roughening the inner stromal surface of the cornea in preparation for the graft in the DSEK procedure. Overall length: 111mm, 4.4 inches.




L. E3124 Terry Scraper Non-Irrigating

Used to gently roughen the inner stromal surface of the cornea in preparation for the graft in the DSEK procedure. Overall length: 115mm, 4.5 inches.

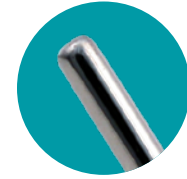


M. E3125 Cindy Sweeper for DSEK

Designed for a smooth glide across corneal surface to smooth out interface between stromal surface and the DSEK graft. Overall length: 112mm, 4.4 inches.

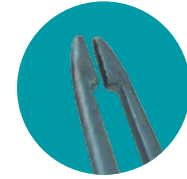

N. ET3122 Charlie Forceps II

A modification to the ET3121 Charlie Forceps, to minimize compression at the tips and maximize donor endothelial survival during donor insertion for the DSAEK, DSEK and DLEK procedures. The forceps have been modified with a 150 micron space between the tips, allowing tissue insertion without crushing. Overall length: 104mm, 4.1 inches.

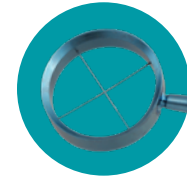

O. ET3126 Corneal Marker 8.5mm/9.0mm

Provides a template for scoring and stripping of Descemet's membrane during the DSEK procedure. Overall length: 132mm, 5.2 inches.



DALK




E0479 Fogla Trifacet Spatula

The trifacet design and its blunt tip permits introduction into the space between Descemet's membrane and the corneal stroma following the formation of the "big bubble" in DALK. The anterior stroma can then be incised over the sharp edge of the spatula in order to enlarge the opening without damaging Descemet's membrane. Overall length: 121mm, 4.8 inches.



E0480 Fogla Pointed Dissector

Designed to facilitate lamellar dissection of corneal stroma allowing the introduction of the Fogla Air Injection Cannula (E4924) for creation of the "big bubble" (plane separation between Descemet's membrane and corneal stroma). The sharp tip and rounded shaft give a more controlled creation of the lamellar pocket in DALK surgery. Overall length: 117mm, 4.6 inches.

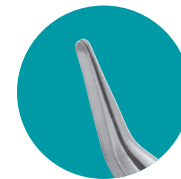

E3210 Fogla DALK Stromal Pocket Forceps

Create a stromal pocket during manual DALK procedures. The rounded tips gently insert into the peripheral corneal stroma. Once inserted, the reverse action jaws spread when squeezed, applying counter traction that allows the edge geometry to separate stromal fibers efficiently. Overall length: 66mm, 3.6 inches.



E3212 Fogla Lamellar Dissector

Make a uniplanar dissection during the DALK procedure. Once stromal tissue is grasped and stretched upwards using a corneal forceps, the dissector is applied to the base of the stromal bed and moved laterally in a windshield wiper motion to help separate the stromal fibers. The semi-sharp tip is designed to maintain a single plane across the stromal bed. Overall length: 123mm, 4.8 inches.



E3217 L Fogla Scissors Left


Remove the residual corneal stroma separated from the Descemet's membrane in DALK surgery. The lower blade of the scissors is thinner and 1mm longer than the upper blade which displaces Descemet's membrane, protecting it from being damaged by the blades. Left. Overall length: 107mm, 4.2 inches.





E3217 R Fogla Scissors Right

Remove the residual corneal stroma separated from the Descemet's membrane in DALK surgery. The lower blade of the scissors is thinner and 1mm longer than the upper blade which displaces Descemet's membrane, protecting it from being damaged by the blades. Right. Overall length: 107mm, 4.2 inches.

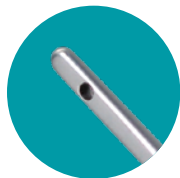


E4924 27 Fogla Air Injection Cannula 27 Gauge

This 27 gauge bottom port cannula is used while injecting air into deep corneal stroma in DALK surgery. The blunt tip helps avoid deeper perforation into the anterior chamber and the flattened cannula is easily inserted into the corneal stromal pocket. The bottom port directs air posteriorly in order to achieve the big bubble. The lamellar track should be created using the Fogla Pointed Dissector (E0480) prior to introducing this cannula. Overall length excluding hub: 18mm, 0.7 inches.



DELIVERING TRUSTED QUALITY FOR OVER 120 YEARS

With over 120 years of designing, developing, supplying and servicing surgical instrumentation, Storz® Ophthalmics and Bausch+Lomb Instruments should be your resource for all of your ophthalmic, ENT and plastic surgery instrument needs. Our comprehensive portfolio of surgical instruments are precision manufactured by instrument craftsmen with over 20+ years of experience. We stand behind our instruments with a guarantee, providing trusted quality that you can rely on across all of your surgical procedures.

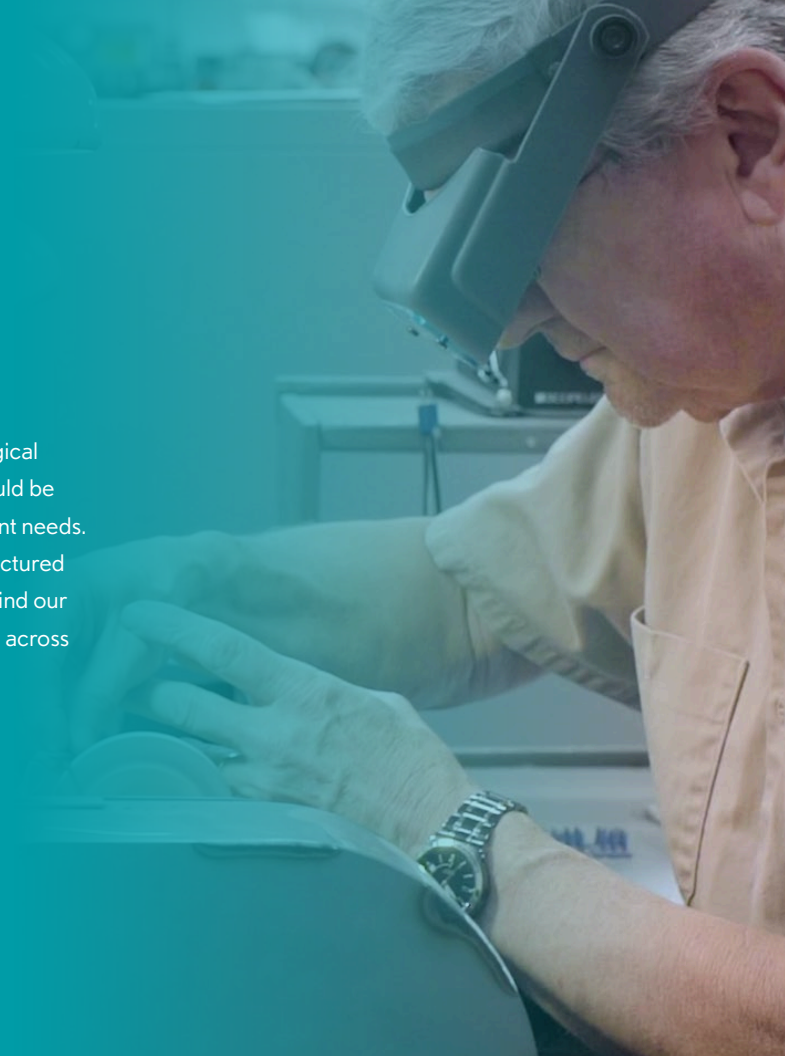

LEARN MORE AT bauschsurgical.com



www.StorzEye.com

INSTRUMENT EXPERTS

Our Surgical Instrument Specialists are at your service to support all of your instrumentation requirements. Bausch + Lomb Storz® Instruments set the standard in Ophthalmic instrumentation for innovative design, quality and service. Our instruments are available directly through Bausch + Lomb Sales and Service professionals.

Our Surgical Instrument Specialists, Instrument DirectConnect Specialists and our Customer Service team are trained to Partner with your Practice to ensure you have access to the best products and the best support possible.

Customer Service (800) 338-2020
Fax (800) 326-7006

Instrument Repairs (800) 658-4751